

SANSA

BY HUXLEY

Quick Start Guide

Scan for **step-by-step video**
and instructions in multiple languages



www.huxleymed.com/instructionvideo

For detailed instructions,
see User Manual (included in kit)




24/7 Support
Call: 888-726-7239

Visit www.huxleymed.com/terms-of-service to view our terms of service

 **Apply SANSA at bedtime.**

1. Prep skin

Perform the following steps in the *left* chest area,
outlined in **blue**.

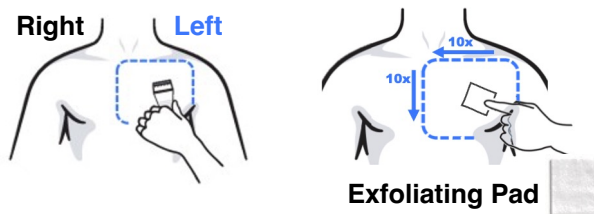

- (A) Wash the area with soap and water.
- (B) Trim chest hair using clippers or razor.
- (C) Gently exfoliate using provided pad.
Wipe skin 10x down and 10x across.
- (D) Clean with provided alcohol wipe.
Allow at least 1 minute to dry.

Flip to other side



⚠ Do not apply without prepping skin (see other side)


2. Apply to skin

- E**  Tear open white pouch and take out SANSa Patch. Tilt SANSa Pod into the Patch and **click** it into place.



- F**  Use the tabs to remove **both ends** of the Patch backing. This will expose the **sticky** adhesive. The device is now ready to be applied.

- G** Using a **mirror** to help with placement, place the SANSa device on the prepared skin area.

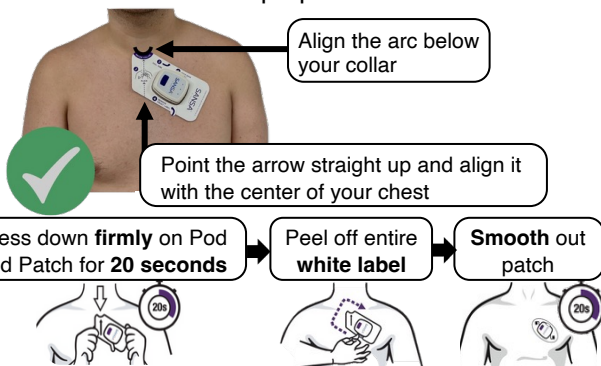

⚠ Once the device is adhered to skin, DO NOT adjust its position.

3. Power on

LBL-10018-02

- H** Turn the Pod on by pressing the **purple button once**. The Pod will vibrate and a **green light** will appear.



⚠ After 5 minutes, the green light will turn off. If orange or red lights appear while wearing the device, please call 888-726-7239.

- I** Setup is complete. **Wear the device to sleep.** For best results, wear for at least **7 hours** or as recommended by your doctor.

4. Next morning, remove & return

- J** Remove the Pod from the Patch by **pressing the latch**. Do not try to turn the Pod off. Simply place back in box.

⚠ It is normal for the Pod to display orange or red lights and vibrate when you remove it.

- K** Seal the box by removing the red liner near the lid. Return it in your **mailbox** or nearest USPS mailbox.

- L** Peel off the Patch **gently** and discard. Wash skin if adhesive remains. Your test is complete.